# SCREEN

## DRUGA TEST

www.screenpharma.it

positivo

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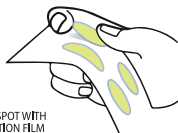
www.screenpharma.it

positive test color

DETACH THE  
DETECTION SPOT WITH  
THE PROTECTION FILM

### Instructions:

- **1** Peel off the protective film before use.
- **2** Take a drop of the liquid with your finger and dab it on the adhesive several times until it is absorbed.
- **3** After 1 minute, if the color of the half part changes, DO NOT DRINK!

The liquid on the spot should be completely dried for the accurate result.

The pattern of the paper may be mistaken for a scratch but it is a characteristic of the paper.

The detection spot is for single use.

Caution: read instructions before use.

Actual picture  
for reference

green

VIDEO INSTRUCTIONS

### POSITIVE AND FALSE POSITIVE

The liquid should be dried completely on the detection sticker for the best result. When the detection sticker matches the "Positive test color", a positive test result has occurred. However, it is recommended that products such as fruit juice, dairy products, and wine do not rely solely on this detection sticker because they may interfere with the reaction of GHB and detection stickers.

### WARNING SKIN AND EYE IRRITANT

After placing a drop of the liquid in to the spot by finger, the liquid remaining on the finger should be washed using tissue or water. If the liquid touches the eyes, rinse it with copious amounts of water. Excess liquid can temporarily stain the clothes. If skin irritation occurs, discontinue use.

### GUARANTEE AND LEGAL NOTICE OF DETECTION RESULTS

A positive result indicates that it may be presence of GHB. However, the positive result may also occur in the case of a false positive. A positive result cannot be the only criterion for determining the presence of the GHB. Neither seller nor any identified third party makes any representations as to the lawfulness of the test results, or as to the reliance upon the results. This detection sticker is warranted to be free from defects under normal use and when stored by the instructions for use. This warranty is in lieu of all other warranties, whether statutory, express, or implied. Distributor shall not be liable for consequential, incidental, special, or any other indirect damages resulting from economic loss or property damages sustained by any customer from the use of this detection sticker. The use of this product constitutes an agreement that no claim shall be made against the seller or any party for damages, including but not limited to compensatory, consequential, incidental, and punitive.