

SCREEN

ALCOL TEST

This test provides preliminary analysis only. You are the only responsible for your decision to drive.

PUT THE SWAB IN YOUR MOUTH AND SOAKING IT WITH SALIVA FOR 10 SECONDS. READ THE RESULT AFTER 4 MINUTES AND COMPARING THE COLOR THAT APPEARS ON THE SWAB WITH THE FOLLOWING COLOR CHART

Color Chart



Alcohol Level

0,00 g/l

0,50 g/l

0,80 g/l

1,50 g/l

Before testing take out the test and bring it to a temperature between 15° and 30°C. Store between 2° and 30°C. Do not put anything in your mouth, including food, drinks, tobacco, chewing gum for at least 10 minutes before testing. Do not read the result after 10 minutes. Keep the test out of the reach of children. Do not dispose of the container in the environment. Indicative qualitative test. Product not intended for medical or diagnostic use.

Distribuito da / Distributed by:


Screen Italia Srl

Via dell'Artigianato, 16

06089 - Torgiano

Perugia - Italia

www.screenpharma.it

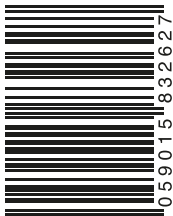

8 059015 832627